

The Klearside® User Manual



Contents List

| | Page |
|-------------------------------|------|
| Contact Information | 1 |
| Declaration Of Conformity | 2 |
| Installation Instructions | 3 |
| Handset Function | 4 |
| Safe Operation of Side Rails | 5 |
| Operating the Side Rails | 6 |
| Cleaning & Disinfection | 7 |
| Service And Inspection Record | 8 |

Contact Information :

BaKare Beds Ltd

Unit 1, Bluewater Estate
Bell Close
Plympton
Devon
PL7 4JH

Sales

Tel : 01752 512222
Fax : 01752 511117

Customer Support

Tel : 01752 512227
Fax : 01752 511117



IMPORTANT

This symbol highlights critical safety information that must be followed



IMPORTANT

This Product is a Medical Device and should not be used without a Risk Assessment in place. The Klearside® side panels should be assessed against any tendency of the end user to push themselves down between the mattress and the Klearside® panel in particular.

The Klearside® Declaration Of Conformity



EU Directive 93/42/EEC

The signatory
BaKare Beds Ltd
Unit 1 ERC
Bell Close
Plympton
Plymouth
PL7 4 JH

Confirms that the products described below and in the form distributed by ourselves meet the basic requirements of Appendix 1 of EU directive 93/42/EEC.

The following Standards apply
EN 60601-1
EN 60601-1-2
EN 60601-2-52
(applicable parts only)

The requirements of the medical products law pertain to the display of a CE seal of approval are thereby fulfilled.

This declaration of conformity becomes invalid if the products are altered without the written agreement of the manufacturer.

Product description

Nursing beds Impulse / Evolution Series
Directive 93/42/EEC of 14/06/1993 concerning medical products. (Appendix I basic requirements). The testing and construction of this product conforms to a Class 1 (Appendix VII) of Medical Product Law of 02/08/1994

Signed

Steven John Baker : CEO Bakare Beds Ltd

Installation Instructions



- Ensure that there is sufficient clearance around the bed so that it will not make contact with any other objects when in operation
- The bed must be used on an even and flat surface
- Ensure that the space under the bed is clear
- Ensure that all cables are free from potential entrapment.
- **IMPORTANT : THE KLEAR SIDE® BED MUST BE USED IN CONJUNCTION WITH THE KLEAR SIDE® MATTRESS TO AVOID ENTRAPMENT ISSUES**

Warranty

The bed and motor system is covered by a 2 Year Parts warranty for manufacture defects. It does not cover damage resulting from abuse or misuse of the bed.

We do not accept liability for damage and disruption caused by operating errors

Maintenance


The bed should be inspected on a regular basis to ensure :-

- That there is nothing trapped within the mechanism of the bed.
- That the functions of the bed operate correctly.
- The wooden slats and holders are undamaged and secure.
- There are no loose fastenings.
- The Klearside® Vinyl sides are correctly tensioned.
- The wheels are free to run and brakes can be operated

At least once a year

- Ensure that all fastenings are tight.
- Unplug the bed from the mains and check for damage along the cables.
- Grease the joins of the lifting bar from the motor.
- The bed is to be inspected by a BaKare technician or a qualified person trained by BaKare.
- The Klearside® Vinyl sides are correctly tensioned

Operating Instructions Handset Function

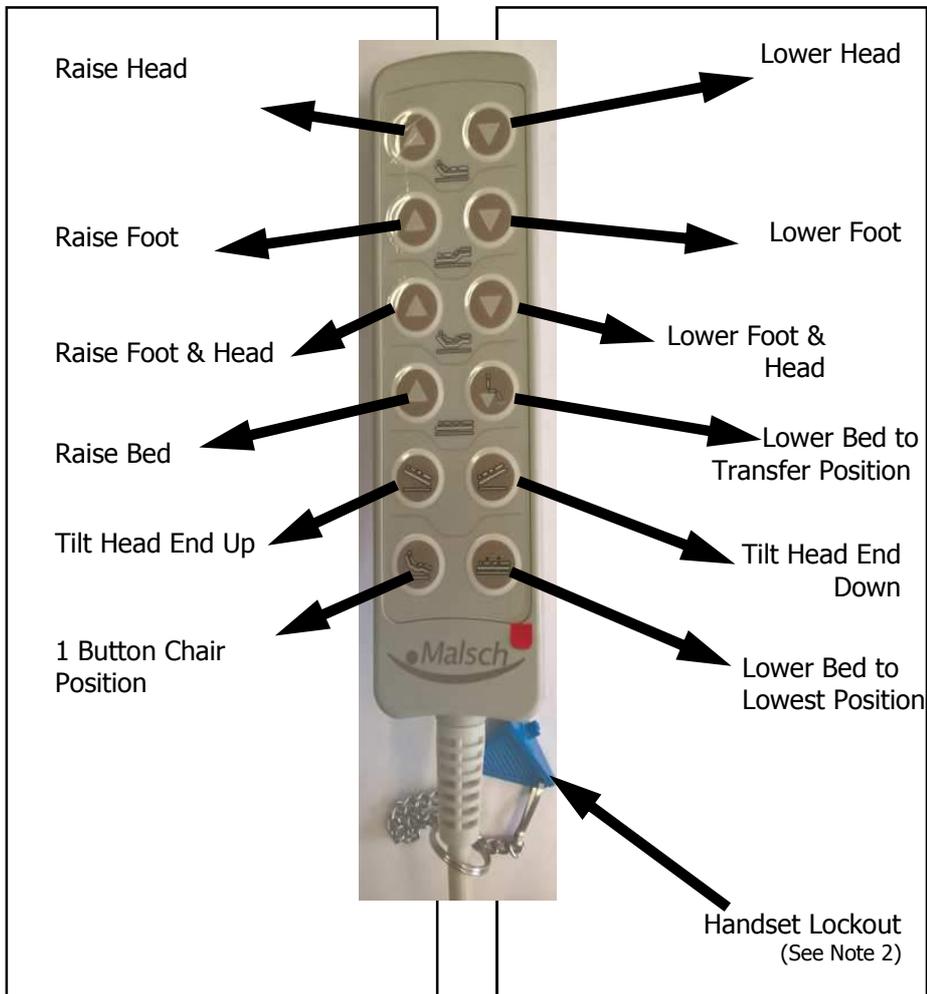

Notes

1. To operate hand controller press and hold the button, once the button is released, the bed will stop moving.
2. The tilt function of the bed is lockable with the blue disc key. In order to operate the tilt function the symbol I must be visible in the window of the disc when inserted in the key hole in the back of the hand control.

Operating Instructions

Safe Operation of The Klearside® Side Rails

1. The side rails should be operated with the bed in its lowest position, this provides the safest position for the occupant of the bed and the easiest position for the carer to operate the side rails.
2. When raising the side rails, first raise the head end of the rails by holding the top rail and lifting (see page 6 for details), when the head end is locked in place proceed to raise the foot end of the rails using the lifting strap to gain extra purchase and allowing the side to become tensioned.
3. When lowering the side rails repeat point 2 in reverse order. Having the head end of the rails raised at all times when adjusting the foot end provides additional protection to the occupant of the bed.
4. Whenever the bed is left occupied and unattended it should be in its lowest position.



Warning— Both ends of the side rail must be raised when the bed is occupied.



Warning— Users are at risk of serious injury if they attempt to climb over the top of the side rails. Always leave the bed in its lowest position when occupied and unattended



Warning— The Klearside® is unsafe for use if a tear occurs on the Klearside.



Warning— Ensure Vinyl is correctly tensioned before use



Safety—Side rails must only be used after a full risk assessment has been carried out.

Operating Instructions

Operating The Klearside® Side Rails


Raising (Head end)



Raising the head-end first means there is reduced risk of occupant falling from bed. To raise the head end side rail:

- 1) Depress the top button on the side of the head board.
- 2) Pull the side rail up until it is higher than the button. Then release the button.

Raising (Foot end)

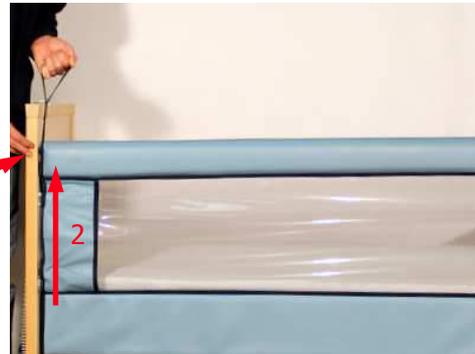

To raise the foot end side rail:

- 1) Depress the top button on the side of the foot board.
- 2) Pull the side rail up using the hand loop until it is higher than the button. Then release the button.

Warning— Both ends of the side rail must be raised.

Attaching Velcro



To further reduce the risk of entrapment, we recommend securing the Klearsides to the head/foot boards using the supplied Velcro.

Lowering

Lowering the side rails means following the previous steps in reverse, starting from the foot end.

Warning— Both ends of the side rail must be lowered.

The Klearside® Cleaning & Disinfection

Wipe Disinfection

Use a wipe Sodium Hypochlorite Disinfectant using no more than 1000 parts per million of Active Chloride

Wooden parts may **not** get into contact with the relevant cleaning agent.

The use of solvents is **not permitted**.

Do not use abrasive cleaners, scouring pads or similar cleaning aids.

Do not use organic solvents such as halogenated/aromatic hydrocarbons and ketone

Please follow these instructions when using cleaning agents and/or disinfectants:

- Decontamination solutions in the prescribed concentrations may not exceed or fall below a pH-value of 6-8.
 - They may not contain any corrosive or caustic substances.
 - They may not contain substances that alter the characteristics of the surface or the properties of plastic or synthetic materials.
 - They may not affect the properties of the lubricants.
 - Water hardness may not exceed 0.9 mmol/l (to 5°d).
- (Fully desalinated water may not be used)

We accept no liability for possible damage to surfaces arising from the use of unsuitable cleaning agents or disinfectants, from the use of incorrect concentrations or from the unsatisfactory care of the bed.

Risk of electric shock, fire or breakdown

Only clean and disinfect the bed after it has been switched off, i.e. when it has been disconnected from the mains power supply (pull plug out of socket).

Protect electrical components from water!

Spray wands

Cleaning and disinfection with spray wands is prohibited.

The beds are not designed to be used with jet stream plants!

*BaKare beds makes every effort to ensure that this document is up to date but does reserve the right to make changes to its products without prior notice.
For the latest information please contact us.*



Service & Inspection Record

The bed is to be inspected by a BaKare technician or a qualified person trained by BaKare every 12 months.

Bed Serial Number _____

| INSPECTION DATE | COMMENTS | ACTION TAKEN | ENGINEERS SIGNATURE |
|-----------------|----------|--------------|---------------------|
| | | | |

BaKare servicing department : 01752 512227
Please call for information on servicing, repairs and extended warranties.

Service & Inspection Recorded